

Simple & Easy Application

- Clean the scar with mild soap & rinse with warm water. Thoroughly dry the scar and surrounding skin area
- Carefully remove Dimiscar gel sheet from the pouch
- Cut the gel sheet to required size
- Remove the plastic liner sheet carefully & without stretching
- Place and position the gel sheet on to the scar
- If required secure with a light bandage

For better Results

- Apply Dimiscar gel sheet initially for 2-4 hours for the first 2-3 days
- For next 2 days extend the use for 48 hours
- Further increase the wearing time to 12 - 24 hours



KRATI
BIOTECH

Healing People

✉ kratibiotech09@gmail.com

☎ 9943928880 / 9629724737