

HyFoam™

Sterile Hydrophilic Polyurethane Foam Dressing A Synthetic Non-Adherent Absorbent Dressing

Product Description

Hydrophilic foam dressing is a sterile synthetic wound dressing, consisting of hydrophilic absorbent polyurethane foam laminated with a low friction water proof polyurethane film, which also acts as a bacterial barrier.

HyFoam™ is also offered as a choice with additional adhesive edges for effective anchorage of the dressing over the wound and high patient compliance.

Indication

- Pressure ulcers
- Leg ulcers
- Venous ulcers
- Arterial ulcers
- Diabetic ulcers
- Highly exuding wounds



SACRUM DRESSING

Application Methodology

- Debride when necessary and irrigate site in accordance with standard protocols.
- Remove excess exudate from surrounding and select a size of the foam dressing that is slightly larger than the wound.
- Apply HyFoam™ in place with a suitable tape to ensure the dressing remains centrally located.
- The foam dressing can remain in site for up to 7 days dependent on patient condition and the level of exudates; initially it may be necessary to change the dressing whenever good wound care practice dictates that the dressing should be changed.
- Remove the dressing using gentle pressure and remove the dressing from the wound bed and discard.
- If required, cleanse the wound site prior to application of a new dressing.

AVAILABLE SIZES

15 X 15 CM

17 X 17 CM

Silicone adhesive border - sacrum dressing



KRATI
BIOTECH
Creating People

info@kratibiotech.com
kratibiotech109@gmail.com
☎ 9942828880 / 9628724737
website: www.kratibiotech.com