

KOLAGRA[™]-GM

Sterile Medicated Resorbable Collagen Dry Sheet with
Gentamicin IP 0.2% w/w

PRODUCT OVERVIEW

Coverage of non- infected wounds with auto grafts has been the best option for preventing fluid loss from burn areas and controlling the infections. However, infected wounds require attention for the use of antibiotics for complete healing. Suitable antibiotics have to be added to manage the infected burn wounds, diabetic wounds, leprosy wounds and other non-healing wounds and ulcers.

KOLAGRA-GM contains collagen from bovine source and coupled with Gentamicin.

KOLAGRA-GM has been found to be a suitable product to meet the long felt need of treating infected open surface areas.

Gentamicin is an aminoglycoside antibiotic used to treat many types of bacterial infections. Gentamicin is a type of aminoglycoside and works by disrupting the ability of bacteria to make proteins. which typically kills bacteria (*Bindu TVL. et al, Trds. Bio. Met. Art.Org, Vol.24, 2010 P. 120.130*).

Wound dressings have been broadly classified as either adherent or non-adherent and absorbent or nonabsorbent. An ideal wound dressing material should have the ability to absorb exudates and toxic components from the wounds surface, maintain a high humidity at the wound/dressing interface, allow gaseous exchange, provide thermal insulation, and protect the wound from bacterial penetration.

KOLAGRA-GM has all the above properties and hence, is used as an effective burn/wound dressing material.

INDICATIONS

- ☞ Superficial, Partial thickness burns, infected burns, & deep dermal wounds.
- ☞ Skin donor sites, skin ulcers (Chronic) - stasis, Graft site, Arterial of Tropical Pressureulcers.
- ☞ Mild to moderately exudate wounds.
- ☞ Cavity wounds.
- ☞ Bedsore of all stages.
- ☞ Dermabrasion.
- ☞ Traumatic loss of skin cover as a temporary skin cover.

KOLAGRA-GM™

Sterile Medicated Resorbable Collagen Dry Sheet with
Gentamicin IP 0.2% w/w

APPLICATION

- Remove **KOLAGRA-GM** from the package and apply it firmly on the raw area for good adhesion, and ensure air bubbles are removed.
- Avoid over stretching the sheet as it may tear the sheet, it can be applied on either surface.

PRECAUTIONS & WARNINGS

- Do not use if the pack is damaged or open.
- This dressing is intended for single use only.
- Potential risk of loss of sterility and infection if sealed pack is opened and reused.
- Avoid using **KOLAGRA-GM** sheet on patients who are hypersensitive to collagen.

AVAILABLE SIZES

10 x 10 cm

10 x 15 cm

15 x 20 cm

15 x 30 cm



KRATI
BIOTECH
Healing People

99439 28880
96297 24737
info@kratibiotech.com
www.kratibiotech.com